

All-In Case Study #1

Surgery Performed by Dr. Gordon Brady, DMD, Oral Surgery Associates, Atlanta, GA

Case Objective


- 1 Extraction of #12 and a surgically guided immediate Implant placement utilizing the ALL-IN Implant solution to load the Implant with a 100% patient specific temporary restoration delivered before the day of surgery.
- 2 Maintain and preserve the Gingiva tissue including the interdental papillae.



Tooth #12 to be extracted and loaded immediately.



Custom temporary abutment and crown placed to an exact fit that provides support to gingiva and interdental papillae.

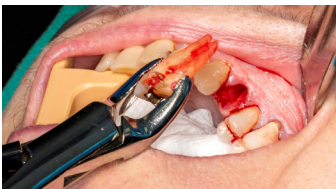

Atraumatic Extraction provides exact extraction socket as planned virtually.



Temporary abutment and crown are delivered with proper interproximal contacts and 1.5mm out of occlusion.



Precision placement of Implant via guided drilling through Surgical guide. Primary Stability achieved.



Precision timing of the Implant created and verified.



Site is grafted using Allograft. No membrane required.

Conclusion

- Greater patient satisfaction.
- Fewer appointments required start to finish.
- Best aesthetic outcomes. No loss of the papillae.
- Shorter post-op healing time due to no sutures and inflammation.